

BIOCREDIT COVID-19 IgG+IgM Duo

SARS-CoV 2 Antibody test


- » The first commercialization Black Gold Particle technology In the world (high sensitivity and specificity)
- » User friendly Dual Color System (Control line: Red, Test line: Black)
- » Rapid Test Time (5~10 Min)
- » Simple detection of SARS-CoV2 antibody

Core Technology Black Gold Particle




Black Gold Particle

Differentiated Black Gold Conjugate Technology

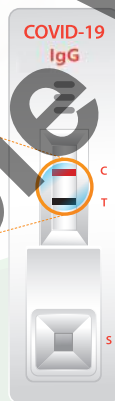


Dual Color System

10 minutes is ok!

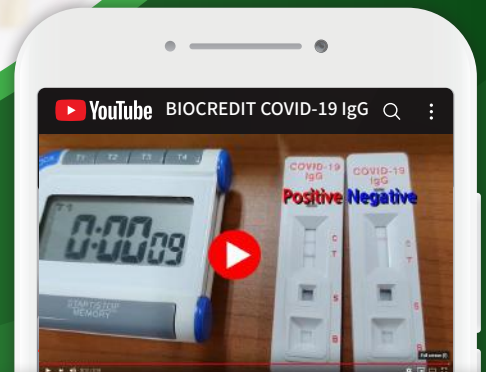

YouTube BIOCREDIT COVID-19

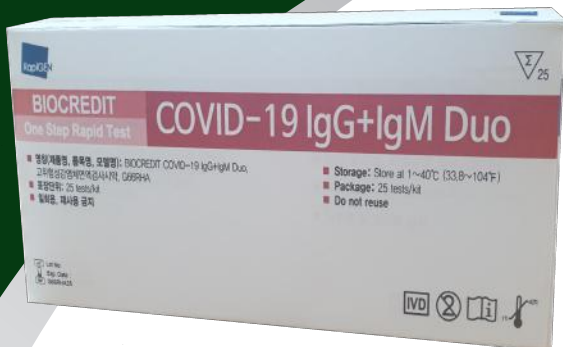

BIOCREDIT COVID-19 IgG+IgM Duo rapid test kit

Intended Use	Detection of SARS-CoV2 antibody
Package	25 Tests / kit
Storage	1~40°C
Specimen Type	Whole blood / Plasma / Serum
Shelf life	12 months from manufacture
Time to result	date 5~10 minute

COVID-19 Rapid Diagnostic Test


PROCEDURE

**IgG and IgM have same procedure.*

Add 10 μ l (serum or plasma) or 20 μ l (whole blood) of specimen into the sample well (S) using provided capillary pipette or micropipette.

Add 3 drops of assay diluent into the buffer well (B) of the device.

Read the result within 5-10 minutes.


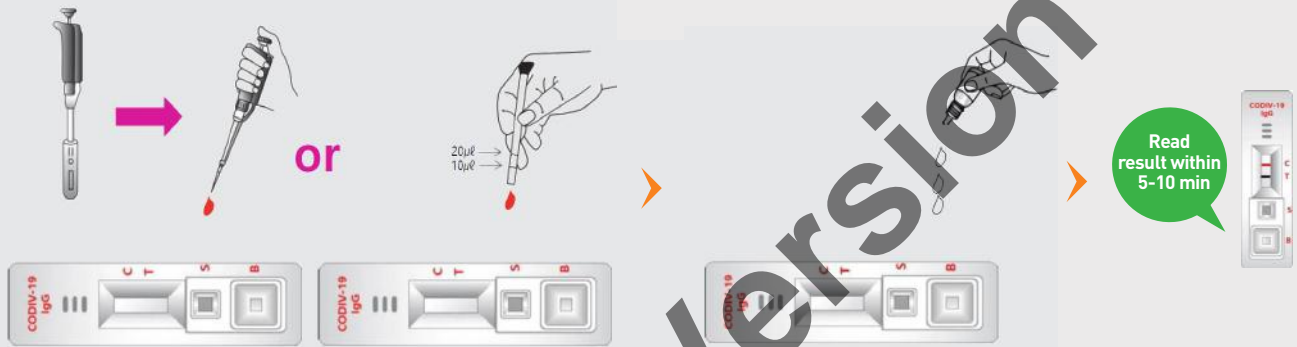


INTERPRETATION

NEGATIVE

POSITIVE

INVALID

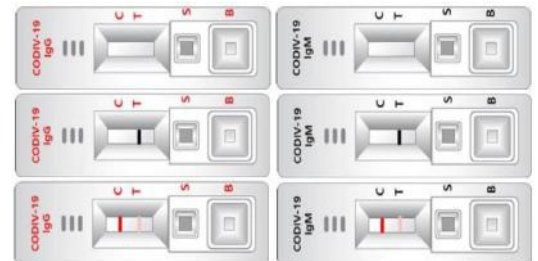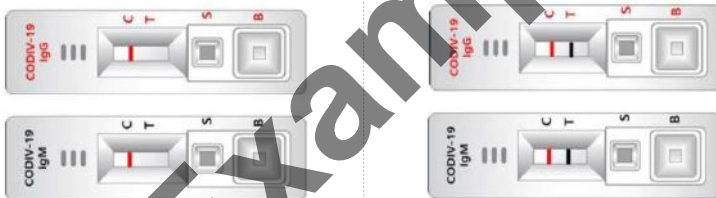

One red line "C" within the result window.

Two bands, **black** To test line and **red** "C" control line within the result window.

No "C" line within the result window. Or test line (T) appears red line faintly. It is recommended that the specimen be retested.

